## **madita., madita-fun. & sofie.**

	Montageanleitung „Kürzung der Fußbankrohre“ .....	2
	Assembly instructions "Shortening the foot support tubes" .....	5
	Instructions de montage „Raccourcissement des tubes du repose-pieds“ .....	8
	Instrucciones de montaje „Recorte de los tubos del banco reposapiés“ .....	11
	Istruzioni di montaggio „Riduzione dei tubi della pedana poggia piedi“ .....	14
	Monteringsanvisning „Avkortning av fotbänksrören“ .....	17

## Anleitung.

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
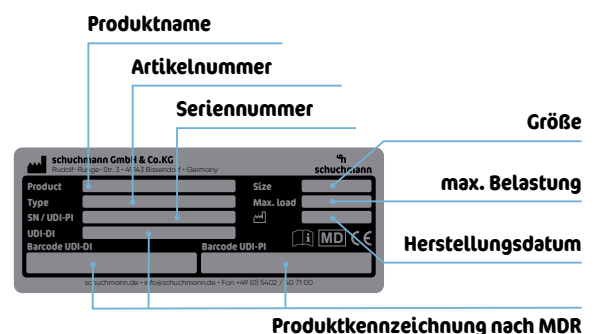
## Instructions.

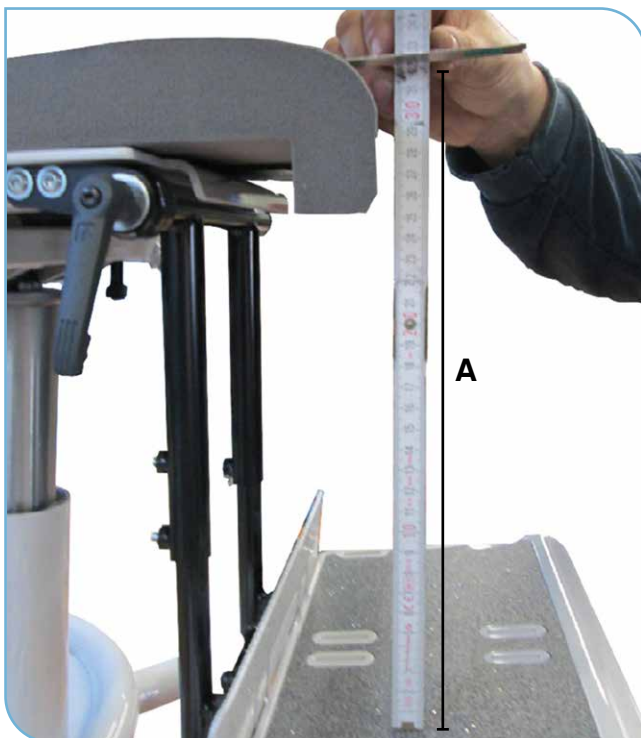The following work steps describe the shortening of the foot support tubes of the **madita.**, **madita-fun.** and the **sofie.** It makes no difference whether the seat is equipped with a standard seat cushion or an anatomically shaped seat cushion.

By default, the foot support tubes can be adjusted to the lower leg length (**A**) from the seat cushion to the foot support

for **Size 0** between 20 and 25 cm,  
 for **Size 1** between 24 and 32 cm,  
 for **Size 1b** between 32 and 46 cm,  
 for **Size 2** between 32 and 49 cm

If the foot support is to be adjusted to a shorter lower leg length the foot support tubes must be shortened. Please use the adjacent size table for orientation.

### Step 1

After the tubes have been dismantled, the inner tube must be marked according to the size table.

### Step 2

After marking the distance to be shortened, saw at the marked positions, possibly at a 90° angle. Please use a hacksaw for this.

Size table:

	Shorten by
<b>Size 0</b>	3 cm
<b>Size 1</b>	4 cm
<b>Size 1b</b>	4 cm
<b>Size 2</b>	6 cm

### Step 3

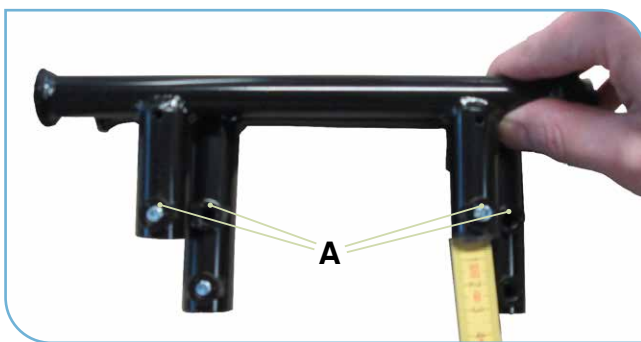
After you have shortened the tubes, the sawing edges must be deburred to minimise risk of injury by tubes with sharp edges. Use either a commercial deburring tool or an iron file.

### Step 4

Now bring the inner and outer tubes together again and mount the foot support at the desired height.

If simple shortening is not enough to reach the desired lower leg length measurement, the tubes can be shortened further. To this end, the inner tubes must be shortened again and this time it is also important to shorten the outer tubes. Refer to the size table on page 5 for the shortening measurements. Subsequently please follow step 5 - 8.

**Shortening to the minimum length is possible, however not recommended, since the foot support tubes are now only fixed with two instead of four locking screws.**

### Step 5

After the tubes have been dismantled, the shortening measurement of the respective basic model size of **madita.**, **madita-fun.** & **sofie.** must be marked on all four tubes. It is mandatory to ensure that the markings on the outer tube are below the upper respective guide slot of the fixing screw (**A**).

### Step 6

After marking the distance to be shortened, saw at the marked positions, possibly at a 90° angle to the respective tube of the four tubes. Please use a hacksaw for this.

### Step 7

After you have shortened the tubes, the sawing edges must be deburred to minimise risk of injury by tubes with sharp edges. Use either a commercial deburring tool or an iron file.

### Step 8

Now guide the inner and outer tube together again and mount the foot support at the desired height.

Summary of possible adjustment lengths for the foot support of **madita.**, **madita-fun.** & **sofie.** in the standard length, for tubes shortened once and in the minimal length:

	<b>Standard length</b>	<b>Tubes shortened once</b>	<b>Minimal length</b>
<b>Size 0</b>	20 - 25 cm	17 - 22 cm	14 - 19 cm
<b>Size 1</b>	24 - 32 cm	20 - 28 cm	16 - 24 cm
<b>Size 1b</b>	32 - 40 cm	28 - 36 cm	24 - 32 cm
<b>Size 2</b>	32 - 49 cm	29 - 43 cm	23 - 37 cm

## General guidelines.

- For reasons of user safety and to maintain product liability, maintenance must be carried out at least once a year by a specialist dealer. The maintenance work carried out must be documented in the maintenance plan of the operating instructions.
- Immediately remove this product from use when any condition develops that might make operation unsafe.
- Use our products and accessories not for any other purposes than those intended. Any other use or use over and above this purpose shall be considered not in accordance with the intended purpose.
- Only use accessories and spare parts made by Schuchmann, otherwise you will endanger the user.

## Spare part supply.

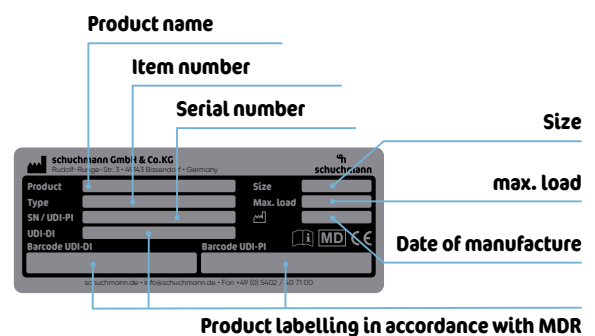
We supply you at any time with required spare parts and, if necessary, provide you with instructions for the assembly or repair of the product. Please note that you are responsible for correct installation or repairs. If in doubt please do not hesitate to contact our customer service as well as our field service for any queries you might have.

All our products are equipped with a type plate on which you find the serial number of the product. Please always state it when ordering spare parts.

You can reach our customer service as follows:

- Phone: +49 (0) 5402/40 71 00
- E-Mail: kundenservice@schuchmann.de

**Only use accessories and spare parts made by Schuchmann, otherwise you will endanger the user.**

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